

DIRECTIONS FOR USE: SURGICAL CAP – DISPOSABLE

Description:

Surgical cap made of non-woven textile is a disposable non-sterile medical device used to protect the operating field from contamination by loose skin particles or hair from medical staff.

Medical devices Class: I non-sterile

Declaration of Conformity:

A copy of the declaration of conformity can be found on our pages www.medicafilter.cz in section „Downloads“ or requested by e-mail: info@medicafilter.cz

Opening:

Open the top lid of the box, or punch the perforated hole if the lid includes it.

Manipulation:

1) Remove the cap from the packaging and check it for damage.

2) Put the cap on your head so that it hides whole head and hair and does not obstruct vision.

Attention: User is responsible for selecting the appropriate cap. If you select and use an unsuitable cap, there may be a risk of insufficient protection and restriction of the scalp, and the cap may not be able to retain falling hair.

3) If the cap gets wet or damaged during the operation, it is necessary to replace it with new one.

4) Dispose in accordance with your internal waste management regulations of group 18 01 04 (wastes whose collection is not subject to special requirements in order to prevent infection) after use.

Expiration:

The shelf life of this medical device is 5 years from the date of manufacture, with appropriate storage.

Storage:

Store at a temperature of 5 °C – 35 °C and at a humidity of 20% - 80%.

Notice:

Do not use the surgical cap if it:

- is damaged after removal from the packaging (holes in the material, torn rubber band, etc.). It is necessary to dispose of and use an undamaged one.
- is past expiry date.
- the packaging is broken before the first opening.
- if you are allergic to latex - applies to AFRO, BARET, CLIP, FLIP, UNI and VARIO caps.


Do not use the product for any purpose other than that specified - see "Description" above.

Do not use the product repeatedly, there is a risk of contamination of the operating field.

The product is not intended to be used as a personal protective equipment.

A thin, breathable material is used to produce of the caps, which can be damaged if subjected to too rough manipulation. Handle gently.

Date: 19.4.2021

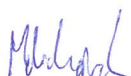

Document number: 333EN0421

MEDICA FILTER spol. s r.o.

Smetanova 421, 341 92 Kašperské Hory, Czech Republic

www.medicafilter.cz

